


microDot


The **ONLY** Vet meter with 20-800mg/dL linear range!
This highly accurate, capillary action loading meter is designed for Canine and Feline testing in the clinic or at home with results in 5 seconds!

Kacey has been providing veterinary glucose meters for 20 years and now presents its Third Generation with enhanced features:

- **Fast:** One Step 5 Second Test
- **Easy:** Simply touch the test strip to the blood site
- **Accurate:** Linear range from 20~800mg/dL (controls available)
- **Ergonomic:** It fits in your hand (3.7"x2"x 0.6") at just 1.7oz

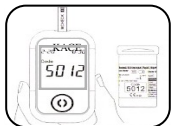

1. Insert the Glucose Test Strip and check code number.

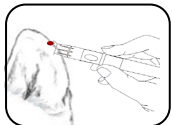

2. Prepare for a drop of capillary blood from the animal's ear.



3. Apply capillary blood sample until the meter beeps.



4. Wait 5 seconds for results to be displayed.



Kit Includes:

- Glucose Meter
- Lancing Device
- 10 Sterile Lancets
- 10 Capillary action test strips
- "Check Code" chip
- Lithium Battery 3V (CR2032)
- Carry case
- User guide

Contact us or your local KACEY dealer for more information about the Kacey Microdot meter.

Authorized Dealer:



KACEY™
DIAGNOSTICS

1854 A Hendersonville Rd
Asheville, NC 28803

Ph: 1(828)685-3569

Fax: 1(828)685-3571

www.KaceyDiagnostics.com

CE

"Kacey Products Providing Better Animal Care"