

KACEY DIAGNOSTICS

Veti-Stx 7™

Veterinary Urine Test Strips

The only seven parameter test of its kind with
PROTEIN/CREATININE RATIO VALUE

Veti-Stx 7 from Kacey Diagnostics is the leader in urine dip technology with seven urine parameters that include Protein and Creatinine ratios. Simply obtain a clean sample of urine, dip and compare to the precision color chart. Its that simple and economical, with 14 months expiration and at up to 50% less than other similar products!

Veti-Stx 7 can test for:

- Creatinine
- Urobilinogen
- Glucose
- Bilirubin
- Protein
- Blood
- pH



Veti-Stx 7 Protein/Creatinine Ratio Values

Creatinine Value	Protein		Readings			
	Trace (10)	30	100	300	2000	
100	0.1	0.3	1	3	20	
500	0.02	0.06	0.2	0.6	4	
1000	0.01	0.003	0.1	0.3	2	
2000	0.005	0.01	0.05	0.15	1	
3000	0.003	0.01	0.03	0.1	0.66	

Normal = Less than 0.5
 Potentially Abnormal = 0.5-1.0
 Abnormal = Greater than 1.0 and less than 5
 Protein Loss = Greater than 6.0

Kacey Diagnostics © 2009

VETI-STX 7

HANDLE

- CREATININE
- UROBILINOGEN
- GLUCOSE
- BILIRUBIN
- PROTEIN
- BLOOD
- pH

A multiple 7 parameter urine strip including the UPC Ratio for the quantitative and qualitative analysis of urine samples.

25 Test Strips

Exp. Date:

Lot #:

For In vitro Use Only
 Store At Room Temperature
 DO NOT FREEZE

KACEY™
 1854A Hendersonville Rd., Suite 112
 Asheville, NC 28803 • 828-685-3569

Tests and Reading Time

Creatinine 60-120 sec. (mg/dL)	Negative	10	30	90	200	350
Urobilinogen 60 sec. (mg/dL)	Normal	2	4	8		
Glucose 60 sec. (mg/dL)	Negative	50	100	250	500	1000
Bilirubin 60-120 sec. (mg/dL)	Negative		1	2	4	8
Protein 60 sec. (mg/dL)	Negative	Trace	30	100	300	2000
Blood 60 sec.	Negative	Trace	Moderate	Trace Hemorrhagic	Small	Medium
pH 60 sec.		5	6	6.5	7	8

UPCR URINE SCREEN GRID (Protein / Creatinine: Ratio Grid)

The UPCR Grid Urine Screen below may be used in obtaining UPC Urine Screen protein / creatinine ratio. The lightly shaded orange areas between the normal and abnormal zones represent questionable or potentially abnormal results. These should be repeated with another method. All abnormal results should be followed-up with quantitative urinary protein / creatinine ratio.

Normal - Less than 0.5 Potentially Abnormal 0.5 - 1
 Abnormal - Greater than 1, and less than 5 Protein Loss - Greater than 6

Product Number: KCVSTX-25 (25 Strips per vial)

In addition to a precision comparison color chart, we will soon offer the Kacey handheld multi test reader compatible with the Vet-Stx 7 strips. Able to read and print results in seconds, it will also read other Kacey tests such as ethylene glycol & organophosphate. Ask about our annual quality control program to assure accuracy...free of charge!

Contact us or your local Kacey Diagnostics dealer for more information about the Vet-Stx 7 urine test strips.

Authorized Dealer:



1854 A Hendersonville Rd
Asheville, NC 28803
Ph: 1(828)685-3569
Fax: 1(828)685-3571
www.KaceyDiagnostics.com

“Kacey Products Providing Better Animal Care”

Kacey Inc. © 2009 All Rights Reserved.